


# Elizabeth Lemon

Case #18498 · Dr. Leo Spaceman · February 15, 2012

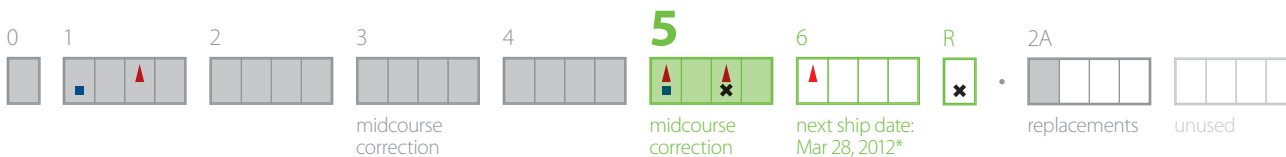

First appointment		Second appointment	
<p>Give steps A &amp; B to your patient to wear for 3 weeks each, and schedule a follow-up appointment in 6 weeks.</p> <p>💧 <b>Check Compliance Checkpoints</b> between these teeth: 4 5 · 11 12 · 28 29</p> <p>▲ <b>Perform IPR</b> between these teeth: 4 5 (0.1 mm) · 28 29 (0.2 mm)</p> <p>■ <b>Place engagers</b> on these teeth: 11 · 12 · 23</p>		<p>Give steps C &amp; D to your patient to wear for 3 weeks each. Don't schedule another fitting appointment until the next phase ships.</p> <p>💧 <b>Check Compliance Checkpoints</b> between these teeth: 6 7 · 11 12 · 28 29</p> <p>▲ <b>Perform IPR</b> between these teeth: 10 11 (0.1 mm) · 11 12 (0.3 mm) · 23 24 (0.3 mm)</p> <p>✖ <b>Remove engagers</b> from these teeth: 1 · 11 · 23</p>	
<p><b>A</b></p> <ul style="list-style-type: none"> <li>5 Rotate Mesially, Translate Lingually, Extrude</li> <li>7 Translate Distally, Rotate Distally</li> <li>8 Translate Distally, Rotate Distally, Extrude</li> <li>23 Intrude, Tip Mesially, Torque Lingually</li> <li>24 Intrude, Tip Mesially, Translate Lingually</li> </ul>	<p><b>B</b></p> <ul style="list-style-type: none"> <li>5 Rotate Mesially, Translate Lingually, Extrude</li> <li>7 Translate Distally, Rotate Distally</li> <li>8 Translate Distally, Rotate Distally, Extrude</li> <li>23 Intrude, Tip Mesially, Torque Lingually</li> <li>24 Intrude, Tip Mesially, Translate Lingually</li> </ul>	<p><b>C</b></p> <ul style="list-style-type: none"> <li>5 Extrude, Rotate Mesially, Translate Lingually</li> <li>6 Translate Distally, Rotate Distally</li> <li>7 Extrude, Translate Distally, Rotate Distally</li> <li>9 Rotate Distally, Translate Mesially</li> <li>10 Intrude, Tip Mesially, Torque Lingually</li> <li>11 Intrude, Translate Linguodistally, Torque Lingually, Rotate Distally</li> <li>12 Extrude, Rotate Mesially, Translate Lingually</li> <li>13 Translate Distally, Rotate Distally</li> <li>21 Extrude, Translate Distally, Rotate Distally</li> <li>22 Rotate Distally, Translate Mesially</li> <li>23 Intrude, Tip Mesially, Torque Lingually</li> <li>29 Intrude, Translate Linguodistally, Torque Lingually, Rotate Distally</li> <li>30 Extrude, Translate Mesially, Rotate Mesially</li> </ul>	<p><b>D</b></p> <ul style="list-style-type: none"> <li>5 Extrude, Rotate Mesially, Translate Lingually</li> <li>11 Translate Distally, Rotate Distally</li> <li>12 Extrude, Translate Distally, Rotate Distally</li> </ul>

\*Do not schedule fitting appointments until the aligners actually ship. These estimated shipping dates are subject to change. Updated shipping information is available at [dr.clearcorrect.com](http://dr.clearcorrect.com).

This treatment plan represents our recommendations based on the treatment you prescribed. Always use your own professional judgement to determine what's best for your patient. | MFG-0010-1.0 | Printed 2/15/2012.